

CLEANING INSTRUCTIONS

FOR MEDICAL PRODUCTS

CLEANABLE KEYBOARDS

- 1. Turn keyboard off
- 2. Use a disinfectant wipe to 'scrub' down the keyboard
 - a. If wipes are not available, than spray disinfectant/cleaner onto a cloth and wipe the keyboard down.
- 3. Allow the keyboard surface to dry/evaporate before powering back on
 - a. This is especially important for the touchpad because it self-calibrates upon power up... any chemicals left on its surface can impact its calibration.

Shown below is a chart listing chemicals that have been successfully tested on TG3's cleanable products.

If the exact cleaner from the chart below cannot be obtained, mild soap and water may be used.

| TG3 TESTED CLEANING PRODUCTS | | |
|---|---|---|
| 10% Hydrogen Peroxide | Ecolab OxyCide™ | Palmolive® Liquid Detergent |
| 3M™ Avagard | Ecolab Quik-Care™ | Phenolic 18 Disinfectant Spray |
| 3M™ Quat Disinfectant 25 | Equate Advanced Healing Hand Lotion | Purell 62% Sanitizer Lotion |
| 70% Isopropyl Alcohol Solution | Equate Antibacterial Wipes | Purell 62% Wipes |
| Aquasonic 100 Gel | Expose® II 256 | PDI Sani-Cloth® HB Germicidal Disposable Wipe |
| Brulin® Maxima 256™ Disinfectant Cleaner | Formula 409® | PDI Sani-Cloth® Plus Germicidal Wipes |
| Caltech Dispatch (.55% Sodium Hypochlorite) | HIBICLENS (Chlorhexidine Gloconate Solution 4.0% w/v) | STERIS' LpH® Disinfectant Cleaner |
| Clorox® Disinfecting Wipes | Household Bleach (Mixed 1:10 with water) | PDI Super Sani-Cloth® Wipes (Alcohol Quat) |
| 3M™ CompuBlend™ IIBase V | Lysol® I.C.™ | T-Spray™ and T-Spray™ II |
| DuPont Virkon | Metrex Metrizyme Dual Enzymatic Detergent | Virex® Tb Disinfectant Cleaner |
| Ecolab Asepti Zyme | Oxivir® Tb | Wexford Labs Wexcide 128 |