
Introduction

These instructions will assist you in replacing the Case Back of your headphone jack equipped Excel Digital or Excel LT™.

WARNING


Please be aware that by installing this kit any applicable warranty is voided unless clearly and explicitly expressed by Crest Healthcare Supply.

Precautions

Please follow these instructions closely to ensure the continued performance of your Excel Digital or Excel LT™ pillow speaker and to preserve the identity of your pillow speaker model.

Instructions

1. Remove the adhesive backing from the sponge and place inside case back as shown in Figure 1.
2. Using a permanent marker write the part number of the original pillow speaker (found on original case back) in the recessed area of the case back as shown in Figure 2. This must be done to ensure your pillow speaker model can be identified for correct parts or replacement.
3. Apply the product label to outside area of new case back as shown in Figure 2.
4. Remove the screws securing the case back to case front from your existing pillow speaker and carefully separate the back case from the front. The headphone jack is attached to a PCB secured to the front case.
5. Note the orientation and routing of the headphone jacks wires around the sponge. Carefully unscrew the threaded ring that holds the headphone jack secure.
6. Insert the headphone jack in through the opening in the new case back and secure with the threaded ring removed in the previous step orientating and routing the headphone jack's wires around the sponge in the same manner as the original. Screw the case back and front together being careful not to pinch any wires.

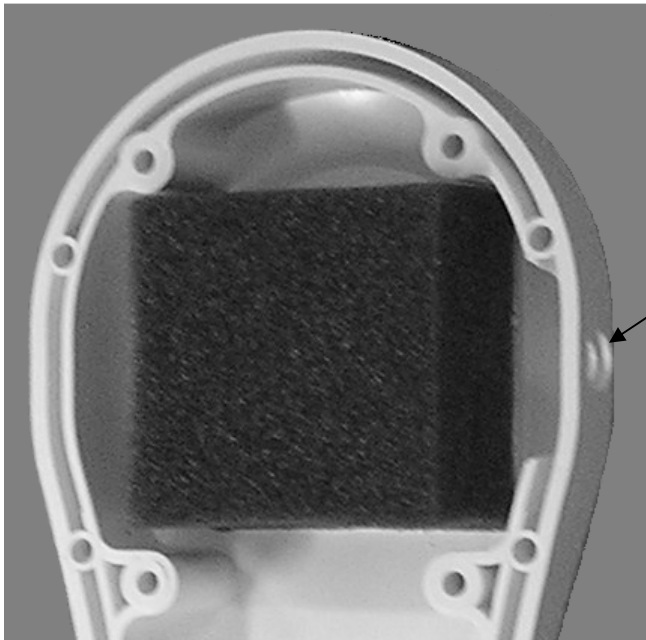

Opening in case back for
headphone jack.

Figure 1



Write part number in
this recessed area
with a permanent
marker

Figure 2

Crest Healthcare Supply
195 S. Third Street
P.O. Box 727
Dassel, MN 55325
1-800-328-8908
1-800-369-9207 (fax)
www.cresthealthcare.com