

Call Crest Technical Support for assistance at: **1-800-328-8908**

www.cresthealthcare.com ©2012 Crest Healthcare Supply



Emergency Pullcord Station, Crest Replacement for Executone[®]

106406 and 106407

Wiring Tables

Crest replacement designation key for Executone models HAV 19ISW41 & HAVJ 189IS/W41

EXECUTONE ORIGINAL TERMINAL DESIGNATION	CREST REPLACEMENT TERMINAL DESIGNATION
1	White/Brown
2	White
3	Green
4	Gray
1A	Violet
2A	Blue
3A	White/Orange
4A	Orange
0	Black
White (lamp common)	Brown
Y	Red/White


Crest replacement designation key for Executone model HAV19IS/W41-BL

EXECUTONE ORIGINAL TERMINAL DESIGNATION	CREST REPLACEMENT TERMINAL DESIGNATION
1	White/Brown
2	White
3	Green
4	Gray
1A	Violet
2A	Blue
3A	White/Orange
4A	Orange
0	Black
White (lamp common)	Brown
Υ	Red/White

Crest replacement designation key for Executone models HPNTES, HPNSES & HPNCBES

EXECUTONE ORIGINAL TERMINAL DESIGNATION	CREST REPLACEMENT TERMINAL DESIGNATION
3	Green
4	Gray
2A	Blue
4A	Orange
0	Black
White (lamp common)	Brown
Y	Red/White

Wiring Schematic

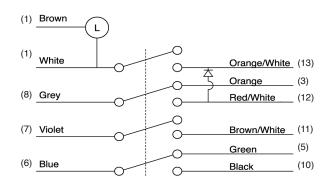

Specifications

ELECTRICAL

Current & Voltage Rating: 4A @ 30V, Bulb 24V

MECHANICAL

 Terminals:
 Brass, silver plated

 Contact Life:
 25,000 cycles minimum

 Mounting:
 Fits standard single gang back box

SIZE MEASUREMENTS

Plate:

4.5" x 2.75" (1 Gang)