

CREST

HEALTHCARE SUPPLY®

www.cresthealthcare.com

CREST

HEALTHCARE SUPPLY®

www.cresthealthcare.com

INSTALLATION INSTRUCTIONS

Emergency Pullcord Station 114433

For Technical Assistance
Call Toll-Free U.S. & Canada

1-800-328-8908

Technicians On Duty

©2008 Crest Healthcare Supply

E-110463



TYPICAL HOOK-UP

1. The red and blue wires are normally open outputs and are usually connected to the corridor light, the nurse station light.
2. The yellow wire is a normally closed output and has current available when the switch is in the off (ip) position. Cap all unused wires.
3. The green and orange wires are connected to incoming power. If only one incoming power line is available connect them together at the time of installation.
4. The brown wire is used for lamp common.
5. Cut off or cap all unused wires.

SPECIFICATIONS

ELECTRICAL

Current and voltage rating: 4 Amps, @ 30 Volts

MECHANICAL

Terminals: Silver plated brass

Life: 25,000 cycles

Mounting: Fits standard single gang back box

POSITION / FUNCTION TABLE

Color	Green	Orange	Blue	Red	Brown	Yellow
Up position	Power	Power	Off	Off	L.C.	On
Down position	Power	Power	On	On	L.C	Off

SCHEMATIC

