

Call Crest Technical Support for assistance at: 1-800-328-8908

www.cresthealthcare.com

©2012 Crest Healthcare Supply



# Pullcord Station, Crest Replacement for Jeron®

2320C, 2331C & 2332C

## **Typical Installation**

- Replace existing station with Crest replacement wire for wire. Note the Wire Color Comparison Table for reference.
- 2. Fasten eyelet on green wire to the backside of the faceplate with mounting screw when screwing the station to the wall box.
- Always test all nurse call stations for correct operation before placing into service.

## **Specifications**

ELECTRICAL Voltage Rating

and Current: 24VDC @ 20mA

**MECHANICAL** 

**Terminals:** Wire Leads

**Mounting:** Screw to standard single gang wall box

**SIZE MEASUREMENTS** 

**Plate:** 4.5" x 2.75" (1 gang)

#### **Wire Color Comparison Table**

Jeron	Crest
White/Orange	White/Orange
Yellow	Yellow
Orange	Orange
Brown/White	Brown/White
Brown	Brown
Black	Black
Green	Green

#### **Schematic**

