



Call Crest Technical Support for assistance at: **1-800-328-8908**

www.cresthealthcare.com

©2012 Crest Healthcare Supply

B-39085
2-15-12
page 1 of 1



Pullcord Station, Crest Replacement for Setco

300-188

Typical Installation

1. Violet and Grey are the normally closed contacts that have power available to them when the switch is in the "Off" position.
2. Yellow and White are the normally open contacts.
3. The unconnected Black wire from the nurse light corresponds with the White wire from the light socket.

Specifications

ELECTRICAL

Current and voltage rating: 1.5A @ 28VDC; 0.5A @ 115VAC

MECHANICAL

Mounting: Fits standard single gang electrical back box

SIZE MEASUREMENTS

Plate: 4.5" x 4.562" (2 Gang)

Function Table

WIRE COLOR	Orange	Violet	Grey	Red	Yellow	White	Blue
Original Terminal Designation	A1	A	D	M	T	F	D1
Up		On	On		Off	Off	
Middle		Off	Off		On	On	
Down (Momentary)		Off	Off		On	On	
Green wire is ground							

Schematic

