



P.O. Box 727 • 195 Third Street South
Dassel, Minnesota • 55325-0727
www.cresthealthcare.com

For Technical Assistance
Call Toll-Free U.S. & Canada

1-800-328-8908

Technicians On Duty

© 2000 Crest Health Care



INSTALLATION INSTRUCTIONS

Economy Pullcord Station

5601



B-39074

SPECIFICATIONS

ELECTRICAL

Current and voltage rating: 4 Amps, @ 30 Volts

MECHANICAL


Terminals: Brass, silver plated.

Life: 25,000 cycles.

Mounting: Fits standard single gang back box.

TYPICAL HOOK-UP

1. The brown & Yellow wires are normally open outputs and are usually connected to the corridor light and the nurse station light.
2. The Blue and Green wires are the normally closed contacts and have current available when switch is in the off (or up) position. Cap all unused wires.
3. The Red and Orange wires are used for incoming power. The Red and Orange wires are usually connected together at the time of installation.
4. Connect one Black wire from the nurse light to lamp common. Connect the other Black wire from the nurse light to the Yellow wire on the switch
5. Cap all unused wires.

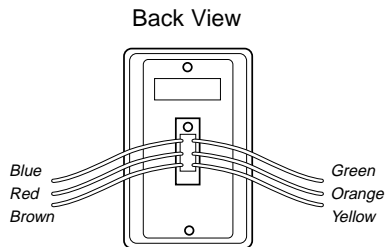

POSITION / FUNCTION TABLE

COLOR	Red	Orange	Blue	Green	Brown	Yellow
Up Position	Power	Power	On	On	Off	Off
Down Position	Power	Power	Off	Off	On	On

ELECTRICAL SCHEMATIC

