

Call Crest Technical Support for assistance at: **1-800-328-8908** 

www.cresthealthcare.com

©2012 Crest Healthcare Supply



# **Crest Deluxe Pullcord Station**

5700

# **Typical Installation**

- 1. The red and blue wires are normally open outputs and are usually connected to the corridor light nurse station light, or buzzer.
- 2. The yellow wire is a normally closed output and has current available when the switch is in the off (up) position. Cap all unused wires.
- 3. The green and orange wires are connected to incoming power and are usually connected together at the time of installation.
- 4. The brown wire is used for lamp common.
- 5. Green is used for "Ground".
- 6. Cap all unused wires.

# **Specifications**

#### ELECTRICAL

**Max Current and** 

Voltage Rating Switch: 4A @ 30V, Bulb 24V

#### **MECHANICAL**

**Terminals:** Brass, silver plated **Contact Life:** 25,000 cycles minimum

**Mounting:** Fits standard single gang back box

#### SIZE MEASUREMENTS

**Plate:** 4.5" x 2.75" (1 Gang)

### **Function Table**

Color	Green	Orange	Blue	Red	Brown	Yellow
Up Position	Power	Power	OFF	OFF	L.C.	ON
Down Position	Power	Power	ON	ON	L.C.	OFF

## **Schematic**

