



Waterproof Pullcord Station

5701
5701CC (With CleanCord)

Typical Installation

1. The red and blue wires are normally open outputs and are usually connected to the corridor light and the nurse station light.
2. The green and orange wires are typically connected to incoming power. The green and orange wires are usually connected together at the time of installation.
3. The brown wire is used for lamp common or (-V).
4. The yellow wire is not used.
5. Cap all unused wires.

Specifications

ELECTRICAL

**Max Current and
Voltage Rating Switch:** .5 A @ 24 VDC
Lamp: 24 VDC / 6 W

SIZE MEASUREMENTS

Plate: 4.5" x 2.75" (Single Gang)

Function Table

Color	Green	Orange	Blue	Red	Brown	Yellow
Up Position	Power	Power	OFF	OFF	(- V) Common	not used
Down Position	Power	Power	ON	ON	(- V) Common	not used

Electrical Schematic

