



## Crest Deluxe Pullcord Station

5702

### Typical Installation

1. The red and blue wires are normally open outputs and are usually connected to the corridor light nurse station light, or buzzer.
2. The yellow wire is a normally closed output and has current available when the switch is in the off (up) position. Cap all unused wires.
3. The green and orange wires are connected to incoming power and are usually connected together at the time of installation.
4. The brown wire is used for lamp common.
5. Green is used for "Ground".
6. Cap all unused wires.

### Specifications

#### ELECTRICAL

Max Current and

Voltage Rating Switch: .5A @ 24VDC

#### MECHANICAL

Mounting: Fits standard single gang back box

#### SIZE MEASUREMENTS

Plate: 4.5" x 2.75" (1 Gang)

### Schematic

