



Single & Dual ¼" Phone Jack Patient Stations, Crest Replacement for Jeron®

P/N 6121C (Single)

Typical Installation

- 1) Review wire designations on the old station.
- 2) Replace the old station with Crest replacement station wire for wire (Review the Wire Color/Function Table for reference.)
- 3) All unused wires must be capped.
- 4) Always test nurse call station for correct operation before placing into service.

Specifications

ELECTRICAL

Voltage Rating:

24 VDC input

MECHANICAL

Terminals:

Wire Leads and Header Connectors

Mounting:

Fits on standard single/dual gang back box

SIZE MEASUREMENTS

Plate:

4.5" x 2.75"

Wire Color/Function Table

Location	Crest Wire Color	Function
P1	Black	Power
	White	GND
	Red	EC
	Blue	A
	Orange	CO
	Yellow	RST

Schematic

