

## Emergency Pullcord Station, Crest Replacement for Dukane®

9A-1405AC




## Technical Specifications

**Plate Size:** 4.5" x 2.75" (1 Gang)

Compatible with Dukane 2000, 2010,  
2020 and Vis-O-Matic.

Non-Compatible with ProCare 2000

## Electrical Schematic

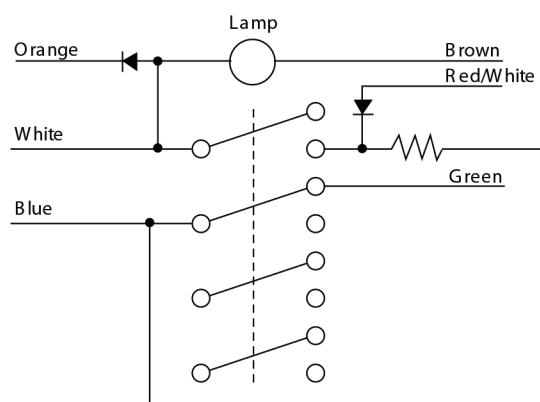

## Function Table

Dukane® Original Wire Colors	Crest Replacement Wire Colors
Red	Brown
Blue/Yellow	Orange
Violet/Yellow	Green
Black	Blue
White	White
Yellow	Red/White

Crest Technical Support: **1-800-328-8908**

Specifications subject to change without notice.  
Visit [CrestHealthcare.com](http://CrestHealthcare.com) for additional technical documents.