



HEALTHCARE SUPPLY®

Call Crest Technical Support for
assistance at: **1-800-328-8908**

www.cresthealthcare.com

©2015 Crest Healthcare Supply

B-39169

Rev. 02

02.10.15

Page 1 of 1



9A3006C



9A3006CC

Shower Pullcord Station, Crest Replacement for Dukane

P/N 9A3006C
9A3006CC

Specifications

ELECTRICAL

Current & Voltage Rating: 24 VDC @ 20 mA

MECHANICAL

Terminals: Wire Leads

Mounting: Screw to standard single gang wall box

SIZE MEASUREMENTS

Plate: 4.5" x 2.75" (1 Gang)

Typical Installation

1. Replace existing station with Crest replacement, referencing the wire color comparison table on right.
2. When installing ensure the o-rings and gasket are correctly used to ensure a waterproof seal.
3. Always test all nurse call stations for correct operation (including cord out if applicable) before placing into service.
4. Hook up new replacement station, matching wire colors.