

Call Crest Technical Support for assistance at: **1-800-328-8908** www.cresthealthcare.com ©2016 Crest Healthcare Supply



Emergency Pullcord Station & Waterproof Emergency Pullcord Station, Crest Replacements for Cornell®

P/N E-104-1C E-104-1WPC E-104-1WPCC

Typical Installation

- 1. Review wire designations on old station.
- 2. Replace the old station with Crest replacement station wire for wire. Review the Function Table below for reference.
- 3. Violet, Blue and Yellow Wires are not used. All of the unused wires must be capped.
- 4. Always test the nurse call station for correct operation before placing into service.

Specifications

ELECTRICAL Voltage Rating: MECHANICAL Terminals: Mounting: SIZE MEASUREMENTS Plate:

24 VDC input


Wire Leads Fits on standard single gang back box

4.5" x 2.75"

Function Table

Cornell Wire Color	Crest Wire Color	Function
Red	Red	24 VDC Positive (+)
Brown	Brown	Annunciator (A)
Grey	Grey	Extra (E)
Black	Black	Negative Input (F-)
Orange	Orange	Looping Input (L)
White	White	Red Light (RL)
Green	Green	Earth Ground (G)
Violet	Violet	Not used (capped)
Blue	Blue	Not used (capped)
Yellow	Yellow	Not used (capped)

Electrical Schematic

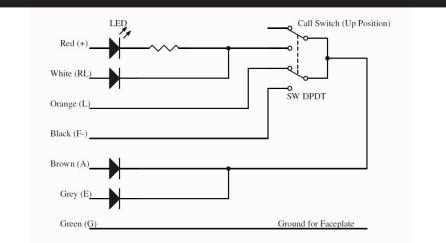

B-39175 Rev. 03 03.02.16 Page 1 of 1