

Call Crest Technical Support for assistance at: **1-800-328-8908** www.cresthealthcare.com ©2016 Crest Healthcare Supply





Bath/Emergency Station,

Crest Replacement for Rauland®

PCS113CC

P/N PBS113C PCS113C (Waterproof) PCS113CC (Waterproof with CleanCord)

Typical Installation

- 1. Replace existing station with Crest replacement wire for wire. Note the Wire Color Comparison Table for reference.
- 2. Disconnect header from original Rauland station and connect it to the Crest Replacement.
- 3. When installing the waterproof pullcord station, ensure the o-rings and gasket are correctly used to ensure a waterproof seal.
- 4. Always test all nurse call stations for correct operation before placing into service.

Specifications

ELECTRICAL Voltage Rating: MECHANICAL Terminals: Mounting: SIZE MEASUREMENTS Plate:

14 VDC @ 100 mA

Wire Leads Fits on standard single gang back box

4.5" x 2.75" (Single Gang)

Wire Color Comparison Table

Pin	Rauland	Crest	Designation
1	Black	Black	-0V
2	Brown	Brown	CLR/CLA
3	Red	Red	+14V
4	Orange	Orange	Zone Light Common
5	Yellow	Yellow	Bath/Code Common
6	Pink	Grey	Annunciator 3000
7	Blue	Blue	FLASH
8	Violet	Violet	Annunciator III