

CLEANING PRODUCT GUIDELINES

We understand the critical importance of disinfecting and sanitizing in healthcare environments.



APPROVED CLEANING & DISINFECTING PRODUCTS

These products are safe for Crest Pillow Speakers, Call Cords and Bed Controls.

- Germicidal disposable wipes
- Germicidal detergent
- Disinfectant cleaner
- 10% bleach
- 70% Isopropyl alcohol
- Other mild detergents



DAMAGING CLEANING & DISINFECTING PRODUCTS

The following chemicals may damage the product. Their use will void the warranty.

- Acetone
- Ammonia
- Aromatic solvents
- Other like chemicals

Also Note

While every attempt has been made to make Crest products as moisture resistant as possible, these devices are NOT submersible.

Crest products are made with durable and rigorously tested materials, however frequent exposure to solvents does degrade casings and cables.

Hydrogen peroxide-based cleaning solutions may cause yellowing to plastics. Related cosmetic issues are not covered by the warranty.


Condition of Warranty

Incorrect cleaning of products will result in the cancellation of warranty terms.

Additional Questions?

Call our friendly Customer Service Team at 800-328-8908.

CREST
HEALTHCARE®

 800.328.8908

 800.369.9207

 CrestHealthcare.com