



Emergency Pullcord Stations, Crest Replacement for Cornell® Systems

E-114-1C
E-114-3C

Typical Installation

1. Replace existing station with Crest replacement wire for wire. Note the Wire Color Comparison Table for reference.
2. Fasten eyelet on green wire to the backside of the faceplate with mounting screw when screwing the station to the wall box.
3. The Red wire with LED is present only in the E-114-3C model.
4. Always test nurse call station for correct operation before placing into service.

Specifications

ELECTRICAL

Voltage Rating: 24VDC input

MECHANICAL

Terminals: Wire Leads

Mounting: Fits standard single gang back box

SIZE MEASUREMENTS

Plate: 4.5" x 2.75" (1 Gang)

Function Table

Cornell Wire Color	Crest Wire Color
Brown	Brown (A)
Grey	Grey (E1)
Black	Black (-)
Orange	Orange (L)
Yellow	Yellow (L2)
Black/White	Blue/White (E2)
Green	Green (E)

Note: Numbers in parenthesis are terminal numbers on the older original Cornell station

Electrical Schematic

