



Crest Code Pink Pullcord Station

114433

Typical Installation

1. The red and blue wires are normally open outputs and are usually connected to the corridor light or buzzer.
2. The yellow wire is a normally closed output and has current available when the switch is in the off (up) position. Cap all unused wires.
3. The green and orange wires are connected to incoming power and are usually connected together at the time of installation.
4. Cap all unused wires.

Specifications

ELECTRICAL

Max Current and
Voltage Rating Switch: .5A @ 24VDC

MECHANICAL

Mounting: Fits standard single gang back box

SIZE MEASUREMENTS

Plate: 4.5" x 2.75" (1-gang)

Schematic

