



Universal Pillow Speaker Nurse Call Latching Station

115009-R1

115009-R2

Technical Specifications


- Electrical:** 24V AC or DC
- Mechanical Mounting:** Fits standard single gang back box
- Plate Measurements:** 4.5" x 2.75" (1 Gang)

Installation Instructions

1. Refer to the Electrical Schematic shown.
2. If using the 7-pin receptacle for pillow speaker applications, see the 7-Pin Receptacle Wiring Table.

Schematics

7-PIN RECEPTACLE WIRING TABLE		
PIN POSITION (PIN VIEW)	WIRE COLOR	FUNCTION
1	RED	LIGHT COMMON
2	BLACK	LIGHT
3	WHITE	TV DATA
4	ORANGE	TV COMMON
5	GREEN	TV AUDIO
6	BROWN	NURSE CALL
8	GREY	NURSE CALL

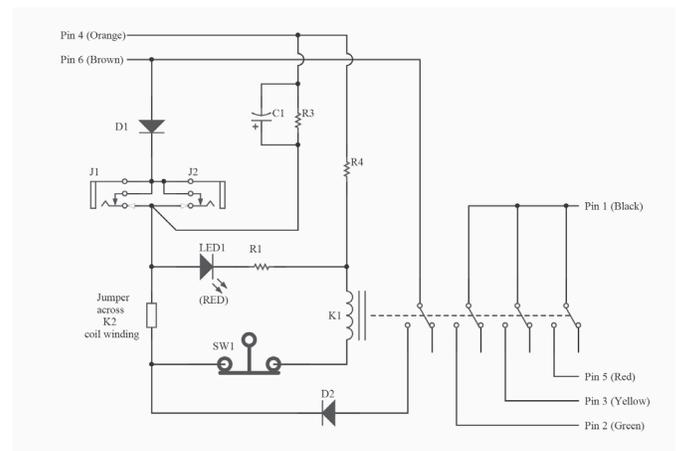

Crest Technical Support: **1-800-328-8908**

Specifications subject to change without notice.

Visit CrestHealthcare.com for additional technical documents.