



Instructions

Locking Call Cord features

- The provided locking call cord will activate the HandiCall when the white plunger is pressed. This will cause the HandiCall to alarm continuously until the call is canceled.
- To cancel the call, simply pull the white plunger back out into the upward position.
- If a momentary alarm is desired, a standard non-locking call cord can be used.
- The HandiCall is equipped with a cord-out feature. When the call cord is removed the HandiCall will alarm continuously until the call cord or a dummy plug is reinserted into the jack.

Two volume settings for the sounder

- The toggle switch on the bottom of the HandiCall switches the HandiCall between two volume levels. The + side indicates the louder of the two settings.

Changing the battery

- Remove the four screws on the back of the unit. The battery will be found inside the HandiCall. Any 9 V battery will work with the unit.

LATEX-FREE

1
YEAR
WARRANTY

Crest HandiCall™ Mobile Call Unit

P/N 115725

Description

This mobile call unit is ideal for use in community spaces, hallways or other facility locations that do not have nurse call functions available. A four foot locking cord is attached to a compact unit that includes a sounder with two volume settings. Unit can be mounted on a stationary chair, wheelchair, set on a table or at one's side. 9 V battery included.