
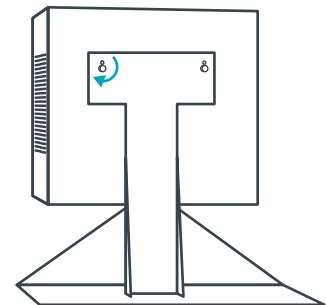
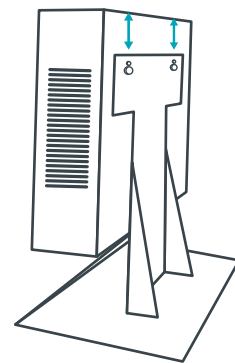


Table Stand for the Protect 900

This table stand has been designed specifically for the Protect 900.
To ensure the safe placement of the unit, please read all safety instructions carefully.

MOUNTING INSTRUCTIONS

1. Place the table stand on a flat surface.
2. Ensure the two parallel holes at the back of the unit are in line with the two parallel screws on the table stand.
3. Fix the unit to the table stand, ensuring a solid connection is made between the screws and the unit.
4. Ensure the unit is level.
5. The securing tabs at the back of the table stand should gently click into position when the unit is mounted correctly, ensuring the unit does not come away from the stand.

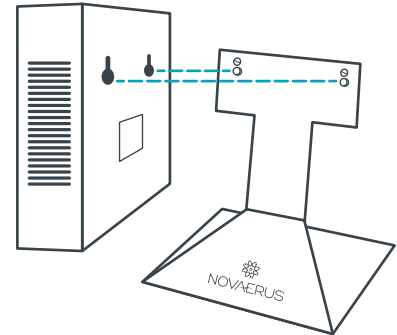

REMOVAL INSTRUCTIONS

1. Ensure the unit is powered off.
2. Gently pull the securing tabs away from the stand and lift the unit up and away from the screws.

