



3080201C



Single 1/4" Phone Plug, Crest Replacement Station for Executone® Nurse Call

3080201C (Stainless Steel)

3080201CPL (Plastic)



B-33505 Red LED
B-33506 Green LED

Description

3080201C/PL patient station comes with 2-way audio communication feature. Uses 1/4" phone plug receptacle for connection to call cord that can be used for call placement. Green Call indicator and red Cancel indicator are provided for visual confirmation.

Typical Installation

- Requires one 9-pin edge connectors (not included, call for part number if needed)
- Review wire designations on the old station
- Replace the old station with Crest replacement station wire for wire
- All unused wires must be capped
- Always test nurse call station for correct operation before placing into service

Technical Specifications

Electrical Rating:	24 VDC input	Terminals:	Edge connectors
Mounting:	Fits on standard double gang back box	Plate Size:	4.5"H x 4.5"W (double gang)

Crest Technical Support: **1-800-328-8908**

Specifications subject to change without notice.
Visit CrestHealthcare.com for additional technical documents.