



Crest Deluxe Pullcord Station

4 Pole, Double Throw (4PDT)

5704

Typical Installation

1. The white/orange, white/red, black and white/brown wires are normally open outputs and are usually connected to the corridor light, the nurse station light, the remote duty station light and a buzzer.
2. The red, orange, yellow and green wires are normally closed contacts which have current available when the switch is in the off (up) position. If these wires are not used in your system, cut them off or cap them.
3. The white, gray, violet and blue wires are connected to incoming power. If only one incoming power line is available connect them together at the time of installation.
4. Cap all unused wires.

Specifications

ELECTRICAL

Max Current and Voltage Rating Switch: 3A @ 30V, Bulb 24V

MECHANICAL

Terminals: Brass, silver plated
Contact Life: 25,000 cycles minimum
Mounting: Fits standard single gang back box

SIZE MEASUREMENTS

Plate: 4.5" x 2.75" (1 Gang)

Function Table

Color	White	Grey	Violet	Blue	Red	Orange	Yellow
Up Position	Power	Power	Power	POWER	ON	ON	ON
Down Position	Power	Power	Power	POWER	OFF	OFF	OFF
Color	Green	Wht/Org	Wht/Red	Wht/Brn	Black	Brown	
Up Position	ON	OFF	OFF	OFF	OFF	L.C.	
Down Position	OFF	ON	ON	ON	ON	L.C.	

Schematic

