



6124C



6125C

## Single & Dual 8-pin DIN Patient Stations with Entertainment, Crest Replacement for Jeron®

P/N 6124C (Single)  
 6125C (Dual)

### Typical Installation

1. Review wire designations on the old station.
2. Replace the old station with Crest replacement station wire for wire (Review the Wire Color/Function Table for reference.)
3. All unused wires must be capped.
4. Always test nurse call station for correct operation before placing into service.

### Specifications

<b>ELECTRICAL</b>	
<b>Voltage Rating:</b>	24 VDC input
<b>MECHANICAL</b>	
<b>Terminals:</b>	Wire Leads and Header Connectors
<b>Mounting:</b>	Fits on standard single/dual gang back box
<b>SIZE MEASUREMENTS</b>	
<b>Plate:</b>	4.5" x 2.75" (Single Gang) 4.5" x 4.562" (Dual Gang)

### Wire Color/Function Table

Location	Crest Wire Color	Function
P1	Black	Power
	White	GND
	Red	EC
	Blue	A
	Orange	CO
	Yellow	RST
P2 and P3	Blue	LSP
	Black	LCN
	Red	LTV
	White	LAX

### Schematic

