



Waterproof Pullcord Bath Station, Crest Replacement for Jeron®

P/N 6132C
 6432C

Typical Installation

1. Review header connector and wire designations on the old station.
2. Replace the old station with Crest replacement station while referring to position "1" for proper orientation of wires/headers.
3. Review the Function Table for reference if needed.
4. All unused wires must be capped.
5. Always test nurse call station for correct operation before placing into service.

Specifications

ELECTRICAL
Voltage Rating and Current:

28 VDC input

MECHANICAL
Terminals:
Mounting:

Wire Lead and Header Connectors
 Fits on standard single gang back box

SIZE MEASUREMENTS
Plate:

4.5" x 2.75" (Single Gang)

Function Table

Location	Position	6132	6432
P1	1	P	P
	2	G	EC/SD
	3	A	S
	4	B	DOT
	5	XA	DIN DIN
	6	Chassis	Chassis

Schematic

