



6311-3JC

6322-3JC

Single & Dual Patient Stations, Crest Replacements for Cornell®

P/N 6311-3JC (Single)
 6322-3JC (Dual)

Specifications

ELECTRICAL	
Voltage Rating:	24 VDC input
MECHANICAL	
Terminals:	Terminal block
Mounting:	Fits on standard triple gang back box
SIZE MEASUREMENTS	
Plate:	6.375"W x 4.5"H (triple gang)

Description

Crest Replacement Bedside or Staff/Duty Stations for Cornell are mounted on a standard 3-gang plate and provide 2-way communication with the master panel station. Standard call cords place a call to the corridor light, master panel and illuminate the station's red LED. When the station is selected at the master panel, the station's green LED lights and a tone sounds to indicate that the caller can speak.

Typical Installation

1. Review PCB designations on the old station.
2. Replace the old station with Crest replacement station. (Review the function table below for reference.)
3. All unused wires must be capped.
4. Always test nurse call station for correct operation before placing into service.

Circuit Board Designator	Function
CS	Remote Cancel Switch Input
+	+24 VDC Input
-	-24 VDC Input
ST	Call Cord Out Signal
A	Annunciator
E	Extra (Same as A)
M	Audio Control
IMC	Audio
PRO	Audio
G	Chassis Ground