



Single 8-pin DIN Patient Station Crest Replacement for Jeron® 8724C


Wire Color/Function Table

Location	Color Wire Color	Function
P1	Orange	Power
	Brown	GND
	White/Orange	A Output
P2	White/Brown	LSP
	Red	LCN
	White	LTV
	Black	LAX
T2	White/Green	B Output
T3	Yellow	GND
T4 & T6	Green	Chassis Ground

Technical Specifications

Voltage Rating:	28 VDC input
Terminals:	Wire Leads and Header Connectors
Mounting:	Fits on standard single back box
Plate Size:	4.5" x 2.75" (1 Gang)

Electrical Schematic

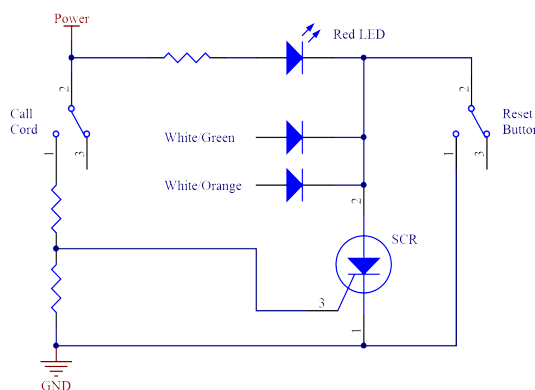

Typical Installation

- 1) Review wire designations on the old station.
- 2) Replace the old station with Crest replacement station wire for wire (Review the Wire Color/Function Table for reference).
- 3) All unused wires must be capped.
- 4) Always test nurse call station for correct operations before placing into service.

Crest Technical Support: **1-800-328-8908**

Specifications subject to change without notice.
Visit CrestHealthcare.com for additional technical documents.