

Single Patient and Shower Station, Crest Replacement for Dukane®

9A2245C, 9A2240C (stainless steel)

9A2245CPL, 9A2240CPL (plastic)

9A2206CC, 9A2206C



9A2245C

9A2240C

Technical Specifications

Current & Voltage Rating:	24 VDC @ 20mA
Terminals:	Wire Leads
Mounting:	Screw to standard single gang wall box
Plate:	4.5" x 2.75" (1-gang)

Typical Installation

1. Replace existing station with Crest replacement, referencing the wire color comparison table below
2. When installing the Shower Station place the o-rings on the mounting screws and center the gasket between the faceplate and the wall to ensure a waterproof seal
3. Always test all nurse call stations for correct operation (including cord out if applicable) before placing into service

Wire Color Comparison Table

Dukane	Crest
Blue/Black	Yellow
Blue	Blue
Blue/White	Green
Yellow/Black	Orange
Violet	Violet
Violet/Red	Grey
White/Red	White
Violet/Green	Brown
Red	Red
Black	Black



9A2206CC

9A2206C

Crest Technical Support: **1-800-328-8908**

Specifications subject to change without notice.

Visit CrestHealthcare.com for additional technical documents.