



9A3006C



9A3006CC

## Shower Pullcord Station, Crest Replacement for Dukane

P/N 9A3006C

P/N 9A3006CC



## Typical Installation

- Replace existing station with Crest replacement, referencing the wiring table below.
- When installing ensure the o-rings and gasket are correctly used to ensure a waterproof seal.
- Always test all nurse call stations for correct operation (including cord out if applicable) before placing into service.

## Technical Specifications

<b>Electrical Rating:</b>	24 VDC @ 20 mA	<b>Terminals:</b>	Wire Leads
<b>Mounting:</b>	Screw to standard single gang wall box	<b>Plate Size:</b>	4.5"H x 2.75"W (1-gang)

Function	Dukane	Crest	Pin #
EXTSENS	White	White	1
SUPERVISION	Black	Black	2
2CLASENS	Green	Green	3
VAN	Brown	Brown	4
CLWINHIBIT	Blue / White	Blue / White	5
FLASH	Yellow	Yellow	6
CLA	Violet	Violet	7
GROUND	Black	Black	8
+VL	Red	Red	9
EXCLR	Blue	Blue	10

Crest Technical Support: **1-800-328-8908**

Specifications subject to change without notice.  
Visit [CrestHealthcare.com](http://CrestHealthcare.com) for additional technical documents.