



BS-8500C

Patient Station Crest Replacement for Rauland®

BS-8500C (Stainless Steel Faceplate)

BS-8500CPL (Plastic Faceplate)


Function Table

Original Terminal Designation	1	2	3	4	5	6
Crest Replacement	Black	Red	N/A	Blue	Yellow	White

Technical Specifications

Current & Voltage Rating DC:	12 VDC
Mounting:	Fits standard single gang back box
Plate Size:	4.5" x 2.75" (1-Gang)

Electrical Schematic

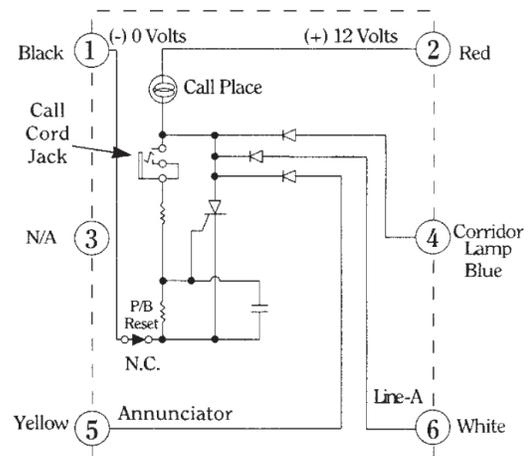

Typical Installation

1. The black wire is connected to the (-) 0V side of the power supply
2. The red wire is connected to the (+) 12V side of the power supply
3. The blue wire is normally connected to the corridor light
4. The yellow wire is normally connected to the annunciator at the nurses' station
5. The white wire is connected to line - A

Crest Technical Support: **1-800-328-8908**

Specifications subject to change without notice.

Visit CrestHealthcare.com for additional technical documents.