



CCP1S-W43C



CCP2S-W43C

Single & Dual 7-Pin Patient Stations, Crest Replacements for Executone®

P/N CCP1S-W43C (Single)
CCP2S-W43C (Dual)

Specifications

ELECTRICAL	
Voltage Rating:	24 VDC input
MECHANICAL	
Terminals:	Edge connectors
Mounting:	Fits on standard triple gang back box
SIZE MEASUREMENTS	
Plate:	6.375" x 4.5" (triple gang)

Description

Single and dual patient stations come with 2-way audio communication feature. Uses 7-pin multi-prong receptacle for connection to pillow speaker or call cord that can be used for call placement. Normal operation or staff assist (emergency) operation for the station can be selected using the slide switch on the PCB. Green Call indicator and red Cancel indicator are provided for visual confirmation.

Typical Installation

1. Requires two 9-pin edge connectors (not included, call for part number if needed).
2. Review wire designations on the old station.
3. Replace the old station with Crest replacement station wire for wire.
4. All unused wires must be capped.
5. Always test nurse call station for correct operation before placing into service.