

# Receptacle Stations, Crest Replacement for Couch™

HA-651C (1-Gang Stainless Steel Faceplate)

HA-651CPL (1-Gang Plastic Faceplate)

HA-651C-01 (2-Gang Stainless Steel Faceplate)

HA-652C (Dual Receptacle, 1-Gang Plastic Faceplate)

**CREST**  
HEALTHCARE®

## Technical Specifications

**Current & Voltage Rating DC:** 24 VDC


**Mounting:** Fits standard single gang electrical back box

**Plate Size:** 4.5" x 2.75" (1-Gang) or  
4.5" x 4.562" (2-Gang)



HA-651C

## Electrical Schematic

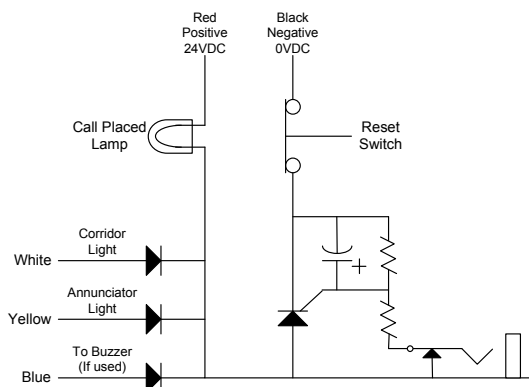

## Function Table

Wire Color	Black	Yellow	White	---	Red	---	---	Blue
Couch Terminal Number	1	2	3	4	5	6	7	8
Function	Negative 0 VDC	To Annunciator Light	To Corridor Light	Remote Button	Positive 24 VDC	No Connection	No Connection	To Buzzer (if used)

## Typical Installation

Connect wires from Crest station to those corresponding to the terminals on the original station. There is no provision for terminal 4 on Crest station. Crest HA-651C requires 1/4" phone plug call cord; Crest #7700W-7 or equivalent.

For two beds sharing the same patient station use double call cords; Crest #7700DW-7 or equivalent.

**Crest Technical Support: 1-800-328-8908**

Specifications subject to change without notice.  
Visit [CrestHealthcare.com](http://CrestHealthcare.com) for additional technical documents.