

Re-Order #	Description	Size	U/M
703460	Legrest with Padded Sides	20"W x 11"D x 8"H	Ea.

Note: The above models fit most 16", 18" and 20" wide wheelchairs.



Fig. 1



Purpose

The Skil-Care Wheelchair Legrest with Padded Sides prevents the legs from falling behind the footrests and also controls the legs from falling off the sides. The padded back and sides also protects the leg from the harsh metal wheelchair components.

Application

1. Before seating resident, lock wheel brakes and rotate footrests to the out-of-the-way position. Assist patient into chair and position him/her as far back on the seat as possible. Reposition footrests.
2. Place the Legrest with Padded Sides on the footrests. Cushion section on footrest, padded backboard with straps against footrest support bars (*Fig. 1*).
3. Place straps behind footrest support bars, secure buckles and tighten straps (*Fig. 2*).
Note that the Skil-Care label is in the upward position
4. Important Safety Note. Remove the legrest and rotate the footrests to the out-of-the-way position when assisting your resident from the chair. Do not permit your resident to stand with footrests and legrest in place.

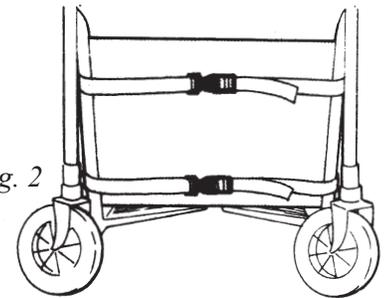


Fig. 2

Easy Care

Skil-Care's Legrest with Padded Sides is made from wipe-clean fabric that can be cleaned with a wipe-clean disinfectant.