



R4K11VC



R4K21VC

Single & Dual 8-pin DIN Patient Stations, Crest Replacement for Rauland® Responder 4000

P/N R4K11VC
R4K21VC

Typical Installation

- Review header connector and wire designations on the old station
- Replace the old station with the Crest replacement station while observing proper orientation of headers.
- Review the Function Table for reference if needed.
- All unused wires must be capped
- Always test nurse call station for correct operation before placing into service.

Technical Specifications

Voltage Rating:	14 VDC input 100 mA	Terminals:	Wire Lead and Header Connectors
Mounting:	Fits on standard single back box	Plate Size:	4.5" x 4.562" (dual gang)

Function Table

Location	Function
TV J1	TV Control & Audio
Left Bed J2	Nurse Call 1
Right Bed J3	Nurse Call 2

Crest Technical Support: **1-800-328-8908**

Specifications subject to change without notice.
Visit CrestHealthcare.com for additional technical documents.