

Pullcord Station

Crest Replacement for Rauland® Responder 4000


R4KPC10C



Technical Specifications

Current & Voltage Rating DC:	0.008A @ 15.5 VDC
Terminals:	6-pin header
Mounting:	Fits standard single gang back box
Plate Size:	5.5" x 5.5" (Oversized 1-gang)

Electrical Schematic

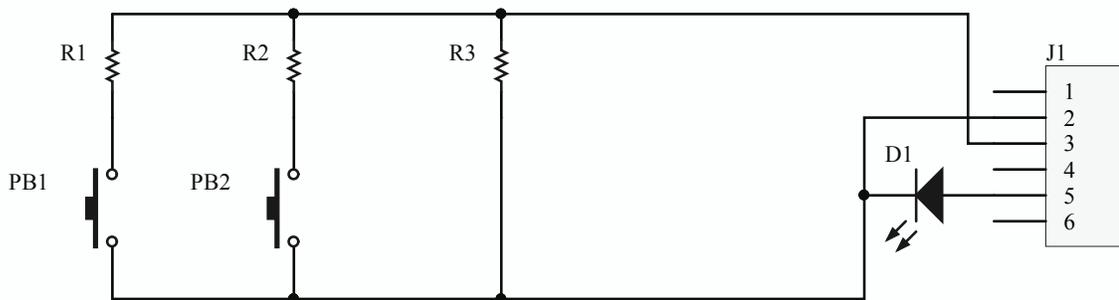

Typical Installation

- 1) Remove old station from wall and unplug the 6-pin harness
- 2) Plug the 6-pin harness into Crest replacement station.
- 3) Mount crest replacement station to single gang box with provided mounting screws.
- 4) Always test all nurse call stations for correct operation before placing into service.

Crest Technical Support: **1-800-328-8908**

Specifications subject to change without notice.

Visit CrestHealthcare.com for additional technical documents.