


UNIVERSAL SAFETY BELT

<u>Reorder #</u>	<u>Size</u>
□620010	72 inches
□620020	60 inches

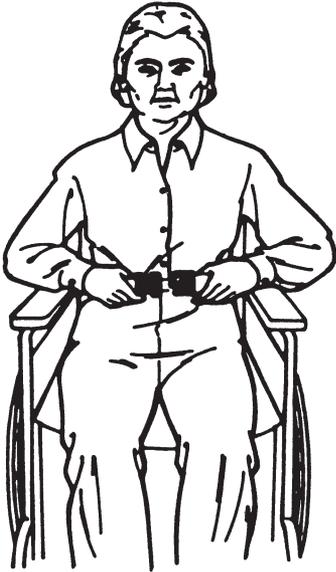

PURPOSE

This multipurpose, resident-release belt may be used as a safety device for wheelchairs, geri-chairs, shower chairs, or commodes. It also serves as a reminder to a resident not to leave the wheelchair without assistance, and as a transfer belt. Fully adjustable, the belt is made from durable, two-inch-wide nylon and features an easy-release buckle.

CAUTIONS

1. This device is not intended to be used as a physical restraint. The easy-release buckle is designed to be released by most residents. The determination as to the appropriateness of the Universal Safety Belt for a specific resident is the responsibility of the facility and its medical authority.
2. If this belt is being used to assist a resident during transfer, the buckle must be placed behind the resident's back in order to limit his/her access to it.

CLEANING

The belt may be cleaned with a spray-and-wipe cleaner.