

Installation of the Sitting Safe Anti-Rollback Wheel Locks (WL080P & WL085P)



STEP 1:

- Remove the old wheel lock with a 10MM wrench, by removing the nylock nut from the inside, underneath the seat.

STEP 2:

- _Install Sitting Safe Anti Rollback Wheel Lock the same way that you removed the old wheel lock.
- Put on the **new** washer and **new** nylock nut but **DO NOT** tighten.

STEP 3:

- Adjust the Sitting Safe Anti Rollback Wheel Lock in the unlocked position where the lever is pulled back.
- Make sure the gap between the tire and foot of the wheel lock is about a quarter inch apart, and then tighten.
- Push the lever forward and make sure the wheelchair **does not** roll.
 - If it does, loosen the nut and adjust the wheel lock closer to the tire. Then re-tighten and make sure wheel lock keeps the chair from rolling in either direction.

STEP 4:

- To apply the anti roll back device: in the unlocked position, flip the black plastic cam lock down. Now the wheelchair will roll forward but not backwards.
- In order to lock the chair in place, flip the cam lock in the up position, push the wheel lock lever forward and the wheel will be locked in place (not rolling forward or backward)



“How To” Video:

https://youtu.be/XNEqHqE7f_0