


## WCP-11 User Instructions

V0.1

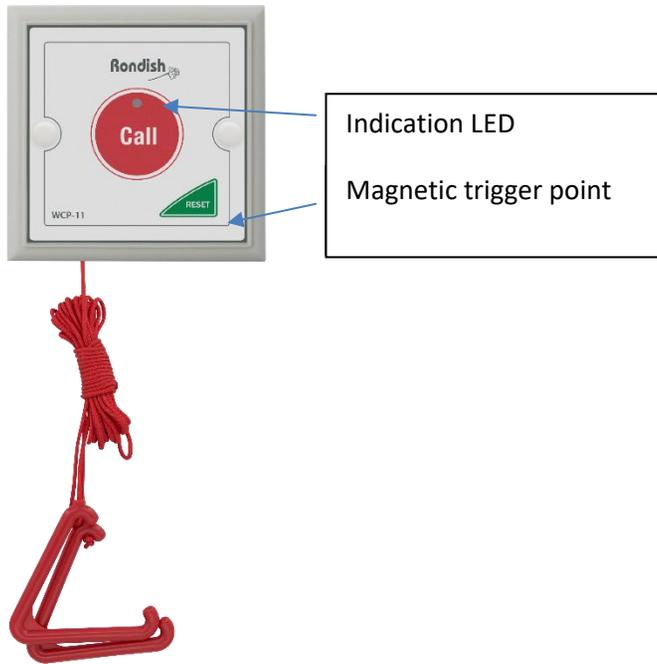

### Activate/Deactivate Tracking

- Apply magnet until the LED begins flashing, then press and hold Reset
- The LED will flash rapidly to indicate tracking is enabled
- The LED will be steady on to indicate tracking is disabled

### Call Trigger

- Press the Call button
- Press Reset to cancel the call

**WARNING:** Risk of explosion if battery is replaced by an incorrect type. Replace only with the same or equivalent type.

**CAUTION:** Lithium batteries cannot be disposed of with normal household waste. For disposal or recycling instructions please contact your local authorities.

## Programming

- Enter programming by applying a magnet to the trigger point and holding Call the indication LED will stay ON when it is ready. When you have completed the input string the WCP-11 will automatically exit programming
- Enter code 28 to change program settings
- Key in 13 digits: t T1T2 D1D2D3D4D5D6 P LLL , where
- t = 1
- T1T2 - Alarm type
  - Alarm - 06 (default)
  - Emergency - 07
- D1...D6 - 6-digit Device ID
- P - Product code
  - Toilet button - 3 (default)
  - Bedhead unit - 8
- LLL - Area code

## Behavior Chart

Call Triggered - includes Call button and pull-cord	Beeps, LED stays ON for 15s
Call Reset	Beeps, LED flashes 3x
Low Battery	LED blinks once every 5s

## Compatibility

Room Indicators	ILB-21 Over Door Light (LoRa)
Call Systems	ZP-01(S) Network Repeater DON-20 Receiver Dongle
Pagers	PAG-11 Caregiver Alert Pager PAG-31 Economy Pager
Nurse Call Interface	CMEX-11 Fixed ¼" Jack CMEX-21 Removable Fly Cable
Accessories	MB-01 Single-Gang Bracket CR2477 Lithium Batteries