



108086

Latching Stations

P/N 108086 108087
108086PL 108087PL

Typical Installation


- 1) Identify your current system by matching it to one of the receptacle styles listed in the chart to the right
- 2) Follow the wiring table to identify which wires on current receptacle terminals attach to respective colored wire in the 108086/PL/108087/PL harness
- 3) Refer to wiring schematic shown, if needed

Specifications

ELECTRICAL	
Voltage Rating & Current	24 VAC or DC
MECHANICAL	
Mounting:	Fits standard single gang back box
SIZE MEASUREMENTS	
Plate:	4.5" x 2.75" (1-Gang)

Plug Style

Identify your current nurse call system by matching it to one of these receptacle styles illustrated.

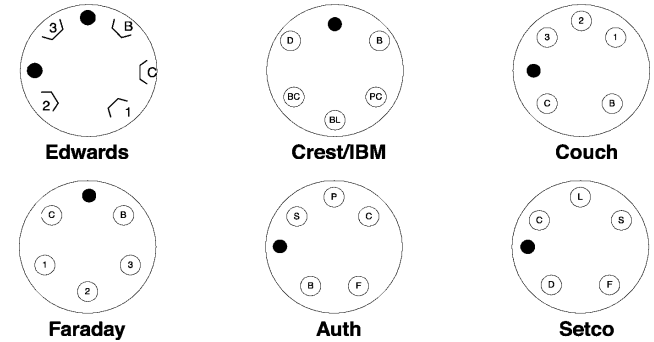

Function Table

for 5-Contact Plug Terminal Designations

WIRE COLOR	CREST	FARADAY	EDWARDS	AUTH	COUCH	SETCO
Brown (hot)	BC	C	C	F	C	F
Red (Buzzer)	B (BZ)	B	B	B	B	S
Orange	CONNECT TO LAMP COMMON WIRE					
Yellow	BL	1	1	C	1	C
Green	PC	2	2	P	2	L
Blue*	D	3	3	S	3	D

*Note: Blue wire in the 108086/108086PL wiring harness should be left unused if possible. If it is necessary to use it as an output, it should only be used to operate an independent lamp circuit, such as the room corridor pilot or annunciator lamp circuit. It should not be used to operate a circuit that is common with any of the other stations in the system, such as a buzzer buss or zone lamp circuit. Call Crest Healthcare Supply Technical Support for clarification.

Schematic

