



ASSEMBLY PROCEDURES FOR

109521-08 & 109521-12

Bed Connector Receptacle Kits

Crest

37 PIN Female

Function

Black	2	Direct Lt. right	CABLE A
White	3	Indirect Lt. right	
Red	20	Direct Lt. left	
Green	21	Indirect Lt. left	
Brown	27	Light common	
Yellow	4	Speaker high right	CABLE B
Orange	7	Nurse Call interlock	
Blue	8	Audio transfer -	
Brown	9	Audio transfer +	
Green	22	Speaker high left	
Red	25	Nurse Call +	
White	26	Nurse Call -	
Black	35	Speaker low common	CABLE C
Blue/Black	5	V.C. wiper Rt.	
Red/Black	6	Radio control right	
Black/Red	10	Interlock	
White/Red	11	Interlock	
Black/White	14	V.C. low common	
Red/White	15	V.C. high common	
Orange	16	Red Nurse ANS. Lt.+	
Red	18	D.C.+	
Green	19	Yellow N. Call Lt. +	
Green/White	23	V.C. wiper left	
Orange/Black	24	Radio control left	
Black	28	Yellow N. Call Lt. -	
White	29	Red Nurse ANS. Lt.-	
Orange/Red	30	Priority bed exit +	
Blue/White	31	Priority bed exit -	
White/Black	33	TV channel stepping -	
Blue	34	TV channel stepping +	
Shield	36	Shield	
Green/Black	37	Radio common	

CREST

HEALTHCARE SUPPLY®

www.cresthealthcare.com

For Technical Assistance
Call Toll-Free U.S. & Canada

1-800-328-8908

B-39060

©2006 Crest Healthcare Supply