

# BREATHCALL® SERIES

JUST A PUFF OF AIR CALLS THE NURSE FOR THE DISABLED PATIENT

[Click here for PDF version](#)

---

## MOUNTING, USE AND CLEANING INSTRUCTIONS

### ***MOUNTING:***

Install clamp on headboard of bed frame so flexible arm projects out and around, pointing toward patients mouth. Point arm slightly downward so liquid from patient's mouth will not leak into tubing.

### ***NORMAL USE:***

When treating patient or changing linen, flexible arm may be flipped out of the way. When patient is back in normal position, BreathCall flexible arm can be moved to original position with straw pointing to patient's mouth.

A sealed package of 12 clear soft vinyl straws two inched long is furnished. To install in the end of flexible arm wet end of straw with alcohol and turn slightly while inserting. Straw can be pulled out for replacement.

Units are tested for and suitable for use in oxygen atmosphere. They are also suitable, due to air activation, for use by patients when *ANY CONTACT WITH ELECTICITY* can present a danger to the patient.

### ***CLEANING:***

The outside and inside of the flexible arm, the clamp and tubing with the exception of the switch plug can be disinfected with most products or mild detergents.

### ***DISINFECTING:***

For disinfecting, one suggestion is, but not limited to, Clorox broad spectrum quaternary cleaner made by [Healthlink, Inc.](http://www.healthlinkinc.net) ([www.healthlinkinc.net](http://www.healthlinkinc.net)).

### ***HOW TO CLEAN AND DISINFECT:***

Always hold, maintain, and keep the wall switch plug higher than the cleansing solution when you immerse the component into your disinfecting solution. After disinfection and while always keeping plug elevated, hang the arm and tubing allowing all cleansing fluids ro empty. Allow adequate time for the unit to dry before raising arm to level of plug tube assembly thus preventing liquid from leading back into the contact plug.

***Caution*** When disinfecting: do not allow any fluids to reach the plug that is inserted into the wall jack or possible malfunction or switch plug may result.

\*\*Note: Accessory package - Replace filters as needed.

### ***\*\*\* Warning \*\*\****

For Healthcare personnel: BreathCall unit is NOT intended for multiple patient use due to bloodborne pathogen concerns and cross contamination.

### ***\*\*\*DO NOT KINK, CRIMP or Bend Tubing.\*\*\****

***\*\*\*Non Electrical\*\*\****

***Needs Air Flow To Activate***

***Note:*** ELECTRICAL PLUG HAS HIGH PRECISION SILVER BALL IN MATCHED SILVER SLEEVE, AND ANY CORROSIVE OR DIRTY LIQUIDS ADMITTED INSIDE WILL CAUSE BALL TO STICK AND SWITCH TO FAIL TO OPERATE.

***Caution:*** Do not steam sterilize unit - high heat will destroy vinyl parts. Maximum temperature is 150 fahrenheit. Do not use sterilizing solutions containing CARBOLIC ACID or PHENOL - These agents after repeated use will harden vinyl parts.

**Electrical:** Unit should be used with maximum rated current of 1/3 amp at 24 volts AC or DC. Higher voltage or current may render the nursecall inoperative.

**Guarantee:** Incorrect use, installation or sterilization of these products may result in cancellation of all warranty terms. All products are unconditionally guaranteed against defects in materials and workmanship. Repair or replacement without charge will be made on products found defective at our factory within one year from the manufacturing date. Freight to our factory is customer's responsibility and return freight will be billed to customer. This guarantee is in lieu of and excludes all other guarantees expressed or implied.