

# Crest Fall Fighter® String Monitor

114053



## Introduction

Congratulations, and thank you for selecting the Fall Fighter® by Crest Healthcare Supply. The Fall Fighter string monitor offers big features in a small package.

- **No on/off switch ensures that the monitor is always functioning.**
- **Multiple volume options allow you to choose high, low or chime tone.**
- **Snap in battery compartment door for ease of use with an optional screw for secure attachment if desired.**
- **Only draws power when the magnet becomes disengaged, saving you money on batteries over time.**
- **Uses common AAA batteries (included).**
- **Easily check battery power with the test button and visual low battery indicator.**
- **For added safety, the magnet connection has been carefully designed to ensure that an individual cannot walk away with the monitor still attached to the string.**
- **Magnet assembly releases from monitor in any direction.**
- **Multiple monitor mounting options.**

## Precautions

**NORMAL USE:** This device should be used to help alert caregivers that a person may be attempting to exit a bed or chair.

**ELECTRICAL:** Use only AAA batteries installed as shown in battery compartment.

**CLEANING AND STERILIZATION:** The outside of the unit may be cleaned with alcohol, most disinfecting and sterilization agents, or mild detergent and water.

**WARNING:** This device should be tested before each use. Failure to do so could result in injury. Use of the Fall Fighter monitor is not a substitute for nursing care. Immediate response to all alarms by a caregiver is necessary.

## Using your Fall Fighter String Monitor

1. Place the magnet assembly into the magnet slot on the front of the monitor.
2. Install the included AAA batteries into the monitor as shown in the battery compartment.
3. Remove the magnet to test the monitor.
4. Select your desired volume position: CHIME/LOW/HI
5. Securely connect the clip on the magnet assembly to resident's clothing.
6. Securely fasten the monitor to stationary object (see Mounting Options below).

## Mounting Options

Several mounting options are included with the Fall Fighter:

- Place the 1" square Velcro® pads directly in the recessed area on the back of the monitor and the other side of the pad on desired location.
- The mounting screws can be used to fasten the monitor bracket to the headboard, headwall or any convenient location.
- The black strap can be used to fasten the monitor bracket to the bed post, wheelchair frame or other places where screw use is not desirable.

## Battery Testing

Testing throughout the life of the batteries is recommended. To test the battery life, simply press the Battery Check button on the face of the Fall Fighter and the green LED will illuminate if battery life is sufficient. The LED will become dimmer as the batteries become weaker.

## FCC Compliant

*This device has been tested to and found in accordance with requirements of FCC Part 15, Subpart B, Class A and Canada ICES-003, Issue: 2012, Class A by TÜV SÜD America, Inc., an independent testing laboratory. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. Modifications not approved by Crest Healthcare Supply could void the user's authority to operate the Fall Fighter.*