

## CableSaver™ Wall Plate Adapter Kit

114707

### Typical Installation

1. After removing your existing plate detach the 37 pin receptacle from your existing mounting. You should not need to cut any wires.
2. Slip the backing plate over the cables coming off of the receptacle. You may attach the receptacle to the backing plate with small flat head screws (not included) if desired. This is not necessary because the design of this plate will secure the receptacle between the back plate and the face plate.
3. Secure the backing plate to your back box using the larger of the two sets of holes in the backing plate.
4. Place the face plate on the back box over the receptacle and put the Cable Saver unit (PN 109533B) on the receptacle.
5. Attach the Cable Saver and the face plate to the backing plate with the screws included with your Cable Saver adapter.

