

Fall Fighter® Infrared Monitor & Replacement Bracket

P/N 115397 & 115572



115397



1
YEAR
WARRANTY

Specifications

Electrical

DC 6 V adapter or 3 AAA batteries

Range

Minimum of 12' directly in front of the sensor

Size Measurements

Case: 3"W x 5"H x 1 3/8"D

Bracket: 1 1/2"W x 5 1/4"H

Instructions

1. Remove the screw from the battery compartment door on the back of the monitor using a Phillips head screwdriver. **NOTE:** *This screw is optional and is not required to hold the battery compartment door in place.* Insert 3-AAA batteries into the monitor in the correct orientation as shown inside the monitor. **NOTE:** *For safety, ensure that the batteries have sufficient power for proper monitoring or plug in power adapter.*
2. Adjust the tone and volume settings on the monitor to fit the user by using the appropriate controls inside the battery compartment. **NOTE:** *If using the "Mute" option, the local alarm will not sound when connected to the nurse call system using a cable.* Mute option does not work unless monitor is connected to the nurse call station using a 1/4" jack.
3. Close battery compartment door. Secure with screw if desired.
4. Mount the monitor by screwing the bracket into the bed frame, wall or other desired location. A hook and loop fastener can also be used (not included).
5. Adjust the monitor within the bracket to ensure the proper angle of the infrared beam.
6. Review the Status LED Functions below and test the monitor to ensure that it is working properly before use.

Status LED Functions

1. **Red flash every 1 second: Warm-Up & Reset Mode**
 2. **Blue flash every 10 seconds: Alarm Armed**
 3. **Red flash in sync with audible beep: Alarm Triggered**
 4. **Continuous red illumination: Low Battery**
 5. **No light: Standby Mode**
7. When the unit is powered on, there is a warm-up period for up to 45 seconds.
 8. To put the unit into standby mode, press and hold the Reset button.
 9. To arm the unit, press the Reset button once. A 15 second delay enables the caregivers to move from the area without activating the alarm.
 10. To stop the alarm from sounding after it has been activated, press the Reset button once to arm it, or press and hold to put it in standby mode.



Call Crest Technical Support for
assistance at: **1-800-328-8908**

www.cresthealthcare.com

©2014 Crest Healthcare Supply

Note:

When battery is low, the red LED light will be lit continuously and will NOT be accompanied by an audible indicator.

WARNING:

The effectiveness of Fall Fighter® Infrared Monitors depend on proper equipment installation and operation. Always test this system prior to use. Fall Fighter® Infrared Monitors are designed to help alert staff of situations that can lead to or result in falls, and are not a substitute for proper caregiver supervision. Please ensure that patients at high risk for falls are sufficiently monitored and help limit environmental hazards that can add to fall risk.