

## Fall Fighter® Seat Belt Alarm

P/N 115406 & 115414

The Crest Fall Fighter Seat Belt Alarm is intended to serve as an alert to staff when a patient at risk for falls unhooks the belt in preparation to ambulate. The Seat Belt Alarm is not intended to be used as a restraint or a substitute for care or supervision.

To operate the seat belt alarm, the belt must be securely bolted to the wheelchair frame and tested before each use.



1 Year Warranty

### Installation Instructions

1. Position the belt across the empty seat, making sure that the blue handle strip is facing out.




2. Bring the end of the belt around to the back of the chair.


3. On the back of the chair, remove the lowest bolt that attaches the upholstery to the chair frame on each side.



4. Put the screw through the seat back, the belt grommet, then into the chair frame screw hole.

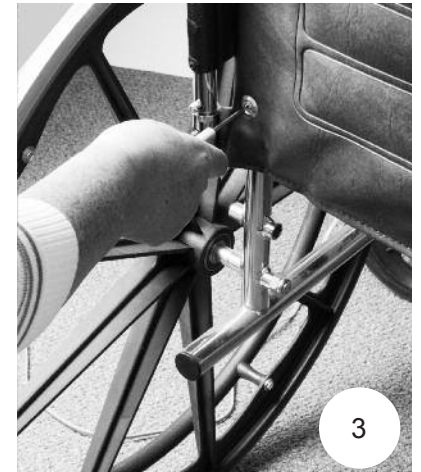

5. Tighten the screw so that the belt is securely attached to the frame.

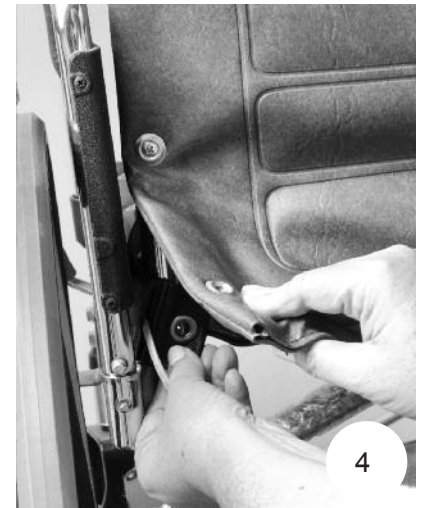

## Installation Instructions Continued

6. Repeat the process on the other side, ensuring that the belt is not twisted at the front of the chair.
7. Ensure that the cord and belt straps are free of the wheelchair's wheels at all times.
8. Attach the Crest Fall Fighter Chair Alarm Monitor, part 114448, or the Crest Fall Fighter Bed Alarm Monitor, part 114449, to the back of the chair and insert the cable into the monitor jack.
9. Test the alarm by ensuring that it sounds when the belt contacts are separated, then reset the monitor to prepare it for use. See Monitor instructions for more detailed testing information.
10. When using the belt with a patient, adjust the belt length for proper fit and comfort.

