



Type Y2 adapter, with one  
9-pin DIN plug on wall end,  
one 9-pin DIN receptacle  
and one 1/4 phone jack  
receptacle

## Crest 9-Pin DIN Y2 Adapter for Rauland® Nurse Call Systems

P/N 115554

### Important Product Instructions

Crest's 9-pin DIN Y2 Adapter for Rauland systems, part number 115554, is designed to be used **ONLY** when both receptacles are in use, either with two call cords, or with one call cord and one dummy plug. This is required because of the nurse call system's ability to cancel a cord-out alarm function. In the event a call cord is removed from the Y2 adapter and the reset button is pressed on the patient station without first replacing the call cord or inserting a dummy plug, the cord out alarm feature will be disabled. This will also potentially eliminate the ability to place a call using the remaining call cord.

As with all call cord and nurse call systems, **ALWAYS** test the nurse call function before use to ensure the system is working properly.

**ALWAYS** make sure the Y2 adapter is set up with both receptacles in use, either with two call cords, or with one call cord and one dummy plug.

The 9-pin DIN Y2 adapter for Rauland systems is **ONLY** intended for use with call cords.