



## Pullcord Station for 24 V Call for Assistance Kit

P/N 115790

### Typical Installation

1. Install Pullcord Station according to the Function Table and Electrical Schematic shown to the right.
2. Fasten eyelet on green wire to the backside of the faceplate with mounting screw when screwing the station to the wall box.
3. Always test the Pullcord Station for correct operation before placing into service.

### Specifications

#### ELECTRICAL

**Voltage Rating and Current:**

24 VDC/VAC @ 20 mA

#### MECHANICAL

**Terminals:**

Wire Leads

**Mounting:**

Fits on standard single gang back box

#### SIZE MEASUREMENTS

**Plate:**

4.5" x 2.75" (Single Gang)

### Function Table

Wire Color	Function
Yellow	Common 1
Orange	Normally Open 1
Brown	Common 2
Brown/White	Normally Open 2

### Electrical Schematic

