



Crest Wireless System Extender, for multiple transmitters

Mounting the Receiver

1. Find a location on the wall where the receiver can reach both a 1/4" jack nurse call station, and an AC power outlet.
2. Mount the receiver to the wall using either the drywall anchors, or the double-sided mounting tape.
3. Power on the unit by plugging AC adapter into the outlet.
4. Plug the 1/4" cable into the nurse call station.

Registering Devices

If you chose to have your Wireless System Extender, for multiple transmitters, pre-programmed by Crest, then it is ready to go out of the box. If not, please follow the instructions below to register devices to your receiver.

The default settings in your Wireless System Extender, for multiple transmitters, are set by Crest. To use the default settings, follow the procedure below (recommended). If advanced programming is required, please call Crest Technical Support for assistance at 1-800-328-8908.

1. Locate the single pry point on the top of the receiver. Insert a flat-head screwdriver and pry the case open.
DO NOT TWIST, it could damage the plastic.
2. Once the receiver case is open, locate the advance button and the program button.
3. Depending on the input number you are programming, this step requires different instructions:
 - a. To register the first device, press the advance button one time.
 - b. To register the second device, press the advance button two times.
 - c. To register the third device, press the advance button three times.
 - d. To register the fourth device, press the advance button four times.
4. Press the program button four times, to scroll through the default settings.
5. The input number will be flashing, to indicate the receiver is ready to accept a device.
6. Reset the transmitter on the device you are registering. Refer to the device's instructions for how to reset the transmitter.
7. Go back to step 3 to register another device.
Note: A maximum of four devices can be registered to one receiver.
8. Once all devices have been registered, snap the plastic cases back together.

Fault Outputs

Along with the primary alarm, there are three types of faults that may trigger a nurse call: a low battery, tamper, or inactivity fault. An inactivity fault will reset itself when the device that was inactive becomes active again. However, the low battery and tamper faults require a reset of the receiver to reset the nurse call. To do this, simply unplug the AC adapter and plug it back in.

Crest Technical Support: 1-800-328-8908

Specifications subject to change without notice.
Visit CrestHealthcare.com for additional technical documents.