



## Wireless Pullcord & Call Cord Stations

### Typical Installation

1. Refer to instructions included with the 116121 receiver while programming your wireless system extender.  
**Note:** Pre-programming options available. Please call Crest Technical Support for assistance at 1-800-328-8908.
2. Remove the battery insulator from the battery compartment on the back of the device.
3. To register the device, press the button labeled "RESET" on the back of the device.
4. Securely screw back plate to wall, if not using existing wall box.
5. Snap the front box assembly to the back plate. Listen for click.
6. Slide front box assembly down to engage the front box hook on the back plate.
7. Tighten set screw located on bottom of station to securely fasten to back plate.
8. Test each station for proper function before use.

### Technical Specifications

<b>Battery:</b>	CR123A 3V
<b>PCB:</b>	UL Recognized, UL94V-0
<b>Plastic:</b>	ABS; UL Recognized, UL94V-0
<b>Front Box Assembly:</b>	5" x 3.15" x 1.5"
<b>Mounting:</b>	Wall, Wall Box

Crest Technical Support: **1-800-328-8908**

Specifications subject to change without notice.  
Visit [CrestHealthcare.com](http://CrestHealthcare.com) for additional technical documents.