

Path-O-Wrap[®]

Medical Surface Barrier for Hospital Beds and Gurneys

PROTECT • PREVENT • PRESERVE



**Protect Your Patient
Protect Your Mattress**

Path-O-Wrap®

Why Path-O-Wrap?

PROTECT Use Path-O-Wrap to protect against damage from mechanical abrasions such as,

- Patients moving on and off mattresses.
- Belt buckles, jewelry, boots and shoes.
- Equipment placed on top of mattresses.

PREVENT Use Path-O-Wrap to prevent damage from cleaning and disinfecting processes.

- Reduce exposure to high concentration disinfectants.
- Reduce exposure to abrasive cleaning procedures.

PRESERVE Path-O-Wrap can potentially extend the life of a mattress.

- Preserve the integrity of the mattress by keeping bodily fluids and other potential dangerous pathogens away from the mattress.
- Reduce premature replacement and costs of a mattress due to damage.

Surface Protection



*unprotected mattress
since 2012*



*protected mattress
since 2012*



*unprotected & protected mattresses
covered with Path-O-Wrap®*

Damaged patient mattresses are common in acute care hospitals. Use Path-O-Wrap as an extra layer of protection to guard against everyday wear and tear, and pathogen contamination.


Path-O-Wrap®

Barrier Protection


- Path-O-Wrap is impervious to bodily fluids and other potentially harmful pathogens.
- Path-O-Wrap has been tested using ASTM methods for blood and viral penetration.
- Path-O-Wrap can be cleaned with hospital disinfectants, thus reducing the use of chemical cleaners directly on the mattress.
- Using Path-O-Wrap is a cost effective solution.
- Use Path-O-Wrap to reduce turnover time between patients.
- Path-O-Wrap can provide protection for gurney mattresses, spinal immobilization backboards, D-MAT cots, Tablets, Defibrillators and more...




Easy to Apply

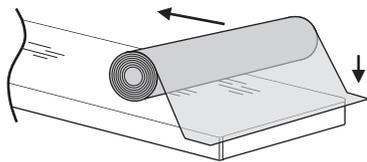


Attach adhesive side down to end of mattress. Gently unroll to end of mattress and adhere.

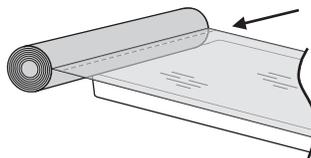

Tear along perforation and remove from roll. Fold down sides and smooth edges.

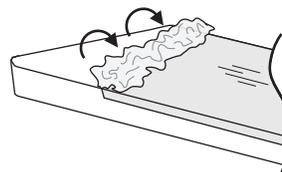

Unpeel and roll from corners while working all the way down mattress until removed.

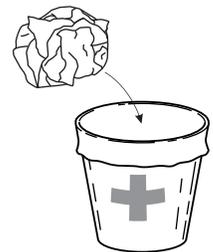

Dispose of according to hospital protocols.

References

- Marks, B, Abboiud, T. *American Journal of Infection Control*. 2016
- Creamer, E. and Humphreys, H. *Journal of Hospital Infection*. 2008
- FDA Safety Communications, April 19, 2013
- Huslage K, Rutala WA, Weber DJ. *Infection Control Hosp Epidemiol*. 2010
- Kramer A, Shewbike I, Kampf G. *BMC Infectious Disease*. 2006
- Ndawula E, Brown L. *Lancet*. 1991

Path-O-Wrap®

PROTECT • PREVENT • PRESERVE



Path-O-Wrap is available in multiple sizes.

Visit www.MedFire.com
for additional information and resources.



4248 Adams Avenue
San Diego, CA 92116
(888) 560-1140